Crescendo

**HEARING
PROTECTION**

INDEX

DRAWINGS (A - K)	3
EN USER INSTRUCTIONS	5
BG ИНСТРУКЦИИ ЗА УПОТРЕБАС	8
CS POKYNY K POUŽITÍ	11
DA BRUGSANVISNING	14
DE BENUTZERHINWEISE	17
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	20
ES INSTRUCCIONES DE USO	23
ET KASUTUSJUHEND	26
FI KÄYTTÖOHJEET	29
FR INSTRUCTIONS D'UTILISATION	32
HR UPUTE ZA KORISNIKE	35
HU HASZNÁLATI ÚTMUTATÓ	38
IT ISTRUZIONI PER L'USO	41
LT NAUDOJIMO INSTRUKCIJA	44
LV LIETOTĀJA INSTRUKCIJAS	47
NL GEBRUIKSINSTRUCTIES	50
NO BRUKERINSTRUKSJONER	53
PL INSTRUKCJA UŻYCIA	56
PT INSTRUÇÕES PARA O UTILIZADOR	59
RO INSTRUCȚIUNI DE UTILIZARE	62
RU ИНСТРУКЦИЯ ДЛЯ ПОЛЬЗОВАТЕЛЯ	65
SK POKYNY NA POUŽÍVANIE	68
SL NAVODILA ZA UPORABO	71
SV BRUKSANVISNING	74

EN

BG

CS

DA

DE

EL

ES

ET

FI

FR

HR

HU

IT

LT

LV

NL

NO

PL

PT

RO

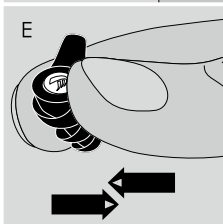
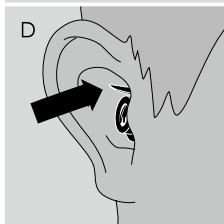
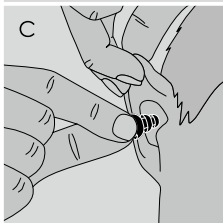
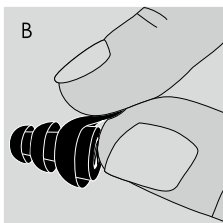
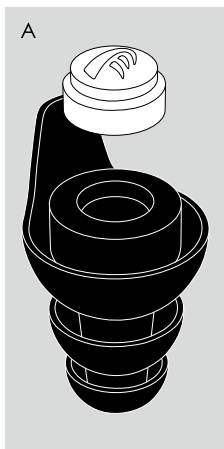
RU

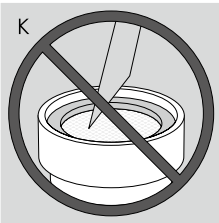
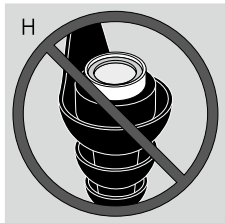
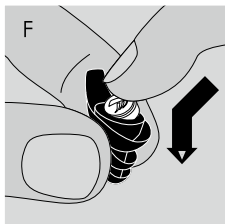
SK

SL

SV

IMAGES (A - B - C - D - E)

EN

BG

CS

DA

DE

EL

ES

ET

FI

FR

HR

HU

IT

LT

LV

NL

NO

PL

PT

RO

RU

SK

SL

SV

The product consists of two parts; the pill shaped filter and the reusable universal fit ear tip (Image A). The filter is fitted into the largest supplied ear tip size. First try inserting this size into the ear canal by following the instructions (1. Inserting into the ear) or by watching this instruction video (QR code).

If the ear tip feels uncomfortable or is difficult to insert, place the filter into a smaller size ear tip (3. Changing ear tip size). Please contact us if it is not possible to achieve a proper seal with the supplied ear tip sizes.

1. Inserting into the ear

1. Ensure the filter is correctly positioned in the ear tip (Image G, H & I).
2. Grip the tongue of the ear tip between thumb and forefinger (Image B).
3. Pull the top of the ear outward and upward to ease insertion (Image C).
4. Position the tongue at the top of the ear making the product easier to remove (Image D).
5. Gently push and twist the product into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal.

2. Removing, Cleaning and Storage

1. Find and grip the tongue of the ear tip (Image D).
2. Remove the product with a slow twisting motion to gradually break the seal with the ear canal.
3. Clean the ear tips after each use. It can be wiped clean with a damp cloth or antibacterial cleaning tissues. We recommend only cleaning the filters if necessary. Do not use alcohol-based cleaning solutions, soaps or detergents.
4. Always store the product in a clean resealable carrying solution. Recommended storage temperature range is between -10 °C and 50 °C.

3. Changing ear tip size

1. Remove the filter from the current ear tip by gently squeezing the ear tip just below the filter (Image E).
2. Ensure that the logo in the filter is facing upward (Image A).
3. Push the filter in at an angle to ease insertion (Image F).
4. Press down on the filter until it is correctly positioned in the ear tip (Image G, H & I).

EN

BG

CS

DA

DE

EL

ES

ET

FI

FR

HR

HU

IT

LT

LV

NL

NO

PL

PT

RO

RU

SK

SL

SV

- If these instructions and warnings are not adhered to, the protection afforded by the product will be severely impaired.
- **CHOKING HAZARD** – Keep the product out of reach of young children.
- The product must be fitted, adjusted and maintained in accordance with the instructions. Improper fitting can reduce its effectiveness.
- In order to achieve the designed attenuation level, it's important that the outside of the ear tip has an airtight seal with the ear canal.
- Performance of the product may be reduced after contact with sharp objects e.g. by puncturing or cutting (Image K).
- Regularly inspect the product to ensure serviceability (Image J).
- The product may be adversely affected when coming into contact with chemicals or alcohol.
- Sudden or fast removal of the product out of the ear canal can damage the eardrum.
- Suitable hearing protection must be worn at all times in noisy surroundings.
- It is not recommended to use the product three (3) years after the production date. The specified attenuation of the product can no longer be guaranteed after expiration.

Dynamic Ear Company (DEC) warrants its products to be free from defects in materials and workmanship for a period of two (2) years from date of purchase. If you believe this product has any defects in materials or workmanship, cease use immediately and contact customer service for help. If a product proves to be defective in material or workmanship, we will repair or replace the product and send it to you at DEC's expense. All returns must include the proof of purchase and documentation specifying the claimed defect. This LIMITED WARRANTY applies only where the products have been properly maintained in accordance with the DEC's instructions and have not been subject to misuse negligence or accident. No returns are to be made directly to DEC unless authorized by the Returns Authorization (RMA) Department.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER WARRANTIES EXPRESS OR IMPLIED, WHICH ARE SPECIFICALLY EXCLUDED, INCLUDING WARRANTIES FOR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL DEC OR SELLER BE LIABLE FOR INCIDENTAL AND CONSEQUENTIAL DAMAGES OR EXPENSES TO THE FULL EXTENT SUCH MAY BE DISCLAIMED BY LAW.

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